# SEMI-MÁSCARA HALF MASK | MEDIA MÁSCARA FFP2 NR

REF: BTHM

INSTRUÇÕES DE USO | WEAR INSTRUCTIONS | INSTRUCCIONES D'UTILIZACIÓN

PT

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EN

1. Hold the mask by the rubber bands, with the nose clip up and the letters out.
2. Put the mask on your face and cover your nose and mouth completely.
3. Attach the elastics to the ears and adjust for comfort.
4. Press the nose clip and adjust it to your face
5. Cover the mask with both hands and breathe quickly to make sure no air is leaking from the sides. If so, adjust until the goal is reached.

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EN

Category III PPE manufactured in accordance with Regulation (EU) 2016/425, for basic use, EN 149: 2001 + A1: 2009 Respiratory protection devices. Filtering half mask to protect against particles. Requirements, tests, marking, in accordance with Certificate nr. 2021EC6835/00/0161 of AITEC, Plaza Emilio Sala nº 1, Alcoy, Spain, Notified Body 0161

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EN: Package of 50 units packed individually in plastic bag. Cartons with 18 boxes each. Design options after Lot number as follows: C - continuous lines; D - non continuous lines.

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EN: This PPE is used to protect against particles, except escape situations

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EN-Keep it dry and ventilated, protect from sunlight and aggressive agents.

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EN-Check the half mask fitting before using. Facial hair will cause leakage problem, if the facial hair is not fully covered, it is unlikely to achieve seal; Air quality (pollutants, oxygen deficiency) must be considered; The use of the equipment in explosive atmospheres is not allowed. The mask does not require maintenance as it is a single-use PPE.

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EN-Do not use it against gases. The user must know the use and handling of the mask. Do not use in areas of risk of explosion. This PPE must not be used against risks other than those described. The filter half mask should not be used for more than one work shift when it is marked "NR" and should therefore be discarded.

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